


PARASITE SUSPENSIONS

CLINICAL • PROFICIENCY • EDUCATION

ILUSTROVANA UPUTSTVA


1

Temeljno promešajte suspenziju energičnim mućkanjem ili kružnim pokretima.

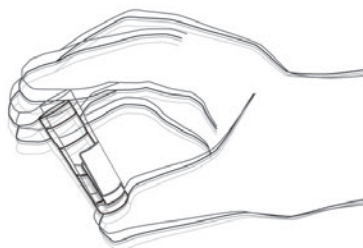

2


Ostavite suspenziju da se taloži 5-10 minuta pre korišćenja u bilo kojoj proceduri testiranja.

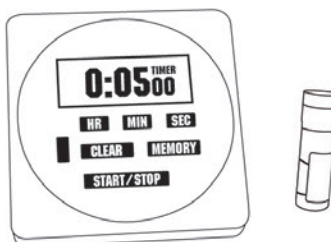

3

Pažljivo ubacite pipetu do dna bočice i uzmite jednu kap za testiranje.

Koristite standardne procedure rada u laboratoriji, reagens za bojenje, ili komplet proizvođača sa uputstvima za testiranje procedura.

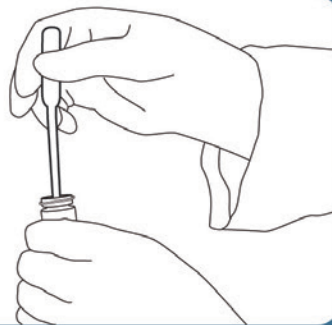

 **Microbiologics®**

A safer, healthier world.